XO ODONTOSON 360

EXTRAORDINARY DENTISTRY

XO ODONTOSON 360 SN|000065

XO ODONTOSON 360

At XO CARE we empower dentists to perform the very best treatments on calm patients, while protecting dentists' health and finances.

XO ODONTOSON 360 is an ultrasonic scaler that redefines the treatment of periodontal disease and daily cleaning procedures. Its unique circular tip movement increases your effectiveness in treating periodontitis and is your guarantee for an enhanced patient experience.

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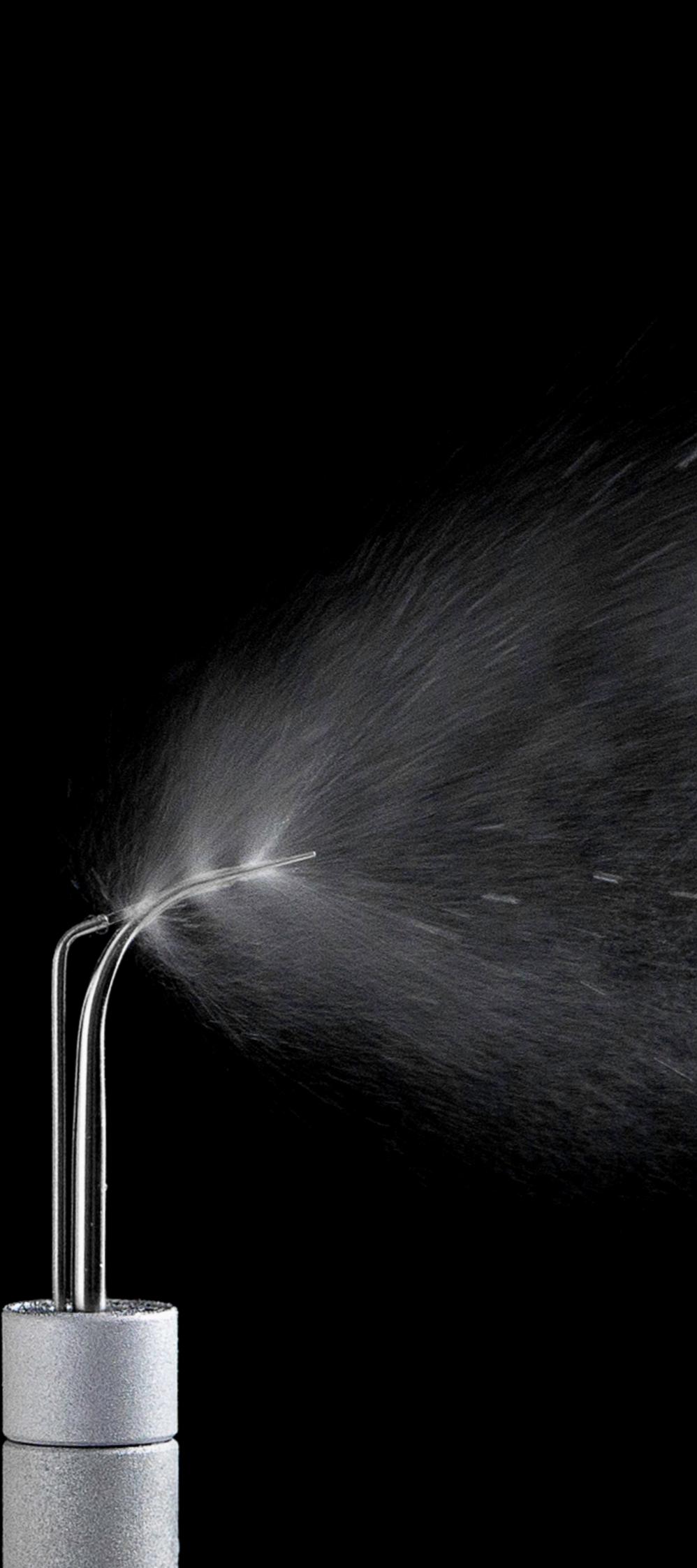
PROFESSIONAL DENTISTRY

XO empowers you to perform the very best dental treatments, enabling you to make full use of your skills and expertise.

Designed to work subgingivally

XO ODONTOSON 360 has been designed to facilitate treatment in areas where your vision is obscured. The scaler instruments are long and thin, allowing you to access furcations and even deepest periodontal pockets.

To provide perfect calculus removal capabilities, especially subgingivally, the instrument oscillates circularly and is active 360 degrees around its tip.

You can supply antimicrobial solution using peristaltic pump installed on your XO unit or integrated into a standalone device.

Successful non-surgical periodontal treatments

XO ODONTOSON 360 will minimize the need for performing a flap surgery. It has been indicated^{1,2} to be particularly effective in connection with subgingival flushing with 0.1% PVP-iodine solution. This non-surgical procedure frequently results in a shallower periodontal pockets and stabilized periodontal attachment.

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¹ Rosling B., Hellström MK, Ramberg P, Socransky SS, Lindhe J. (2001) The use of PVP-iodine as an adjunct to non-surgical treatment of chronic periodontitis. Journal of Clinical Periodontology, 28(11), 1023-31.

² Forabosco, A., Galetti, R., Spinato, S., Colao, P., & Casolari, C. (1996). A comparative study of a surgical method and scaling and root planing using the Odontoson®. Journal of Clinical Periodontology, 23(7), 611-614.

CARING DENTISTRY

We understand and recognize how important it is that your patient remains calm throughout treatment. To this end, XO's equipment is designed to be safe and comfortable and to help you gain the complete confidence of your patient.

The gentlest scaling possible

With XO ODONTOSON 360 your patients experience less pain, even deep subgingivally. The instrument operates at a very high frequency, yielding an exceptionally small amplitude. The instrument tip oscillates circularly and will never hammer against a tooth³.

Damage to the tooth surface is minimized as all instruments are handcrafted from titanium, a material softer than enamel.

Gain patients' trust

When using XO ODONTOSON 360 you are sure that you provide the gentlest and most comfortable experience for your patients. It will not remain unnoticed.

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
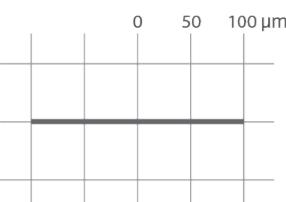
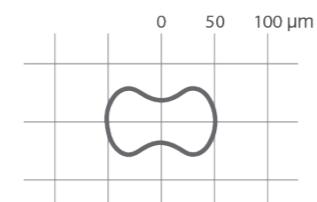
Magnetostrictive Metal Stack

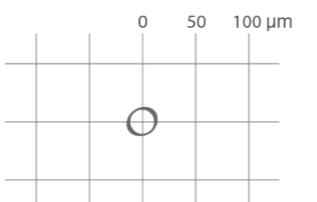XO ODONTOSON 360
Magnetostrictive Ferrite Rod

Comparison of tip movement and amplitude of three different scaler technologies.

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ERGONOMIC DENTISTRY

Your health is one of our main priorities when we design our solutions. We aim to ensure that you sit well, see well and are able to work well – all at the same time.

Tailored to your way of work

Traditional piezo scalers, with a linear tip movement, require you to keep the same parallel position of instrument all around the tooth. This frequently forces you to move your wrist into an unnatural position and in the long run may lead to injuries such as carpal tunnel syndrome. XO ODONTOSON 360 is active all around the instrument tip and allows you to maintain an ergonomic hand positon throughout the whole treatment. The handpiece and its suspension are lightweight and do not strain your wrists and arms.

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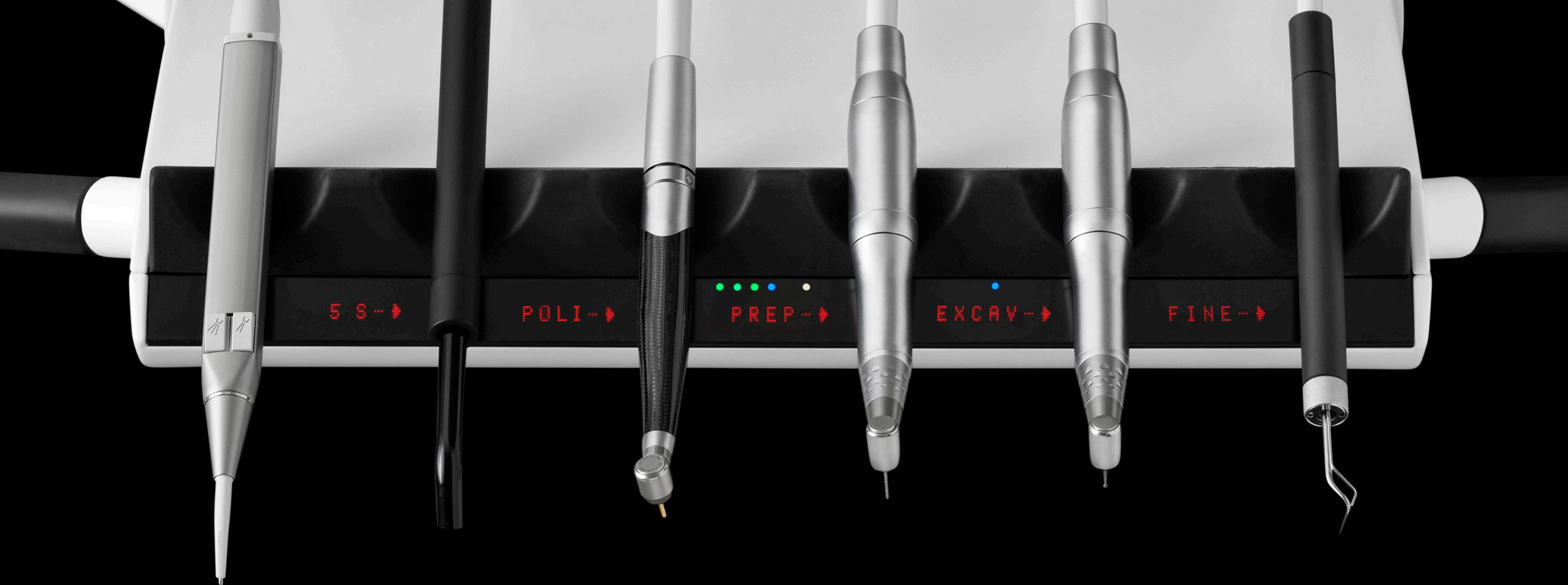
PRODUCTIVE DENTISTRY

Our objective is to help you increase the value of your equipment investment by saving you time and money, giving you maximum uptime and providing an attractive working environment.

Versatile instruments

With XO ODONTOSON 360 you significantly reduce the number of instrument changes. Thanks to circular tip oscillation, you can do more with one scaler instrument.

Faster workflows

Even when you need to change an instrument, you can do it instantly with pull/push instruments. No need to screw instruments into a handpiece.

Similarly, when you want to switch from e.g. lower to upper jaw, simply twist the instrument in the handpiece and carry on with the procedure.

With XO ODONTOSON 360 installed on your XO unit, you can work even more efficiently with XO SMART LINK and instrument programs.

As an alternative, you can use a standalone model with integrated peristaltic pump – see page 16.

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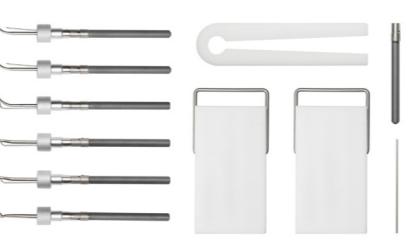
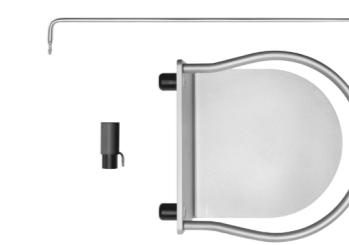
Instruments and service kit supplied with
XO ODONTOSON 360

Kit for external irrigation
XO ODONTOSON 360

XO ODONTOSON 360

standalone model

Ultrasonic scaler with:

- Magnetostriuctive oscillation principle (ferrite rod)
- Two-dimensional circular instrument tip movement
- Small amplitude of instrument tip: 10 – 20 µm
- High frequency: 42.000 Hz
- Instruments consisting of a hand-crafted titanium tip fitted with an exchangeable ferrite rod
- Pull/push instruments

Operation:

- Switch on a handpiece

Irrigation:

- Peristaltic pump (flow rate 10–80 ml/min)
- Switching between sterile saline or antimicrobial solution.

Asepsis:

- Handpiece, hose, handpiece holder and instruments may be autoclaved at 135°C
- Antibacterial powder coat of the control box

Including:

- Electronic control box
- Handpiece with hose (black)
- Integrated peristaltic pump with tube fixture
- Irrigation bag hanger
- Support for liquid reservoir
- Spear for liquid reservoir
- Disposable irrigation kit for XO ODONTOSON 360 standalone model – 2 pcs
- Power cable

Supplied with six instruments fitted into two autoclavable instrument holders in Teflon:

- Universal instrument (209080) – 2 pieces
- Perio instrument (209030)
- Thin Line Straight instrument (209034) – 2 pieces
- Heavy Duty Straight instrument (209010)

Service kit with:

- Ferrite (209100)
- Steel pin (209300) and clamp wrench (309300) for tightening or exchange of ferrite

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


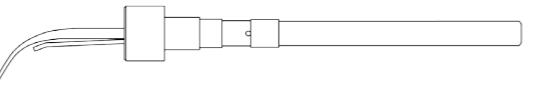
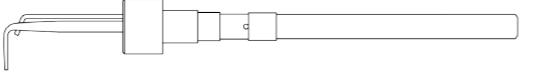
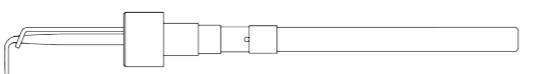
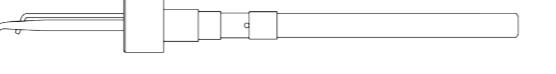
INSTRUMENTS FOR XO ODONTOSON 360:





PROPHYLAXIS

Heavy Duty straight 209010	Particularly useful for removal of supra- and subgingival calculus labially and lingually.		
Heavy Duty right 209012	For removal of supra- and subgingival calculus, in particular in the 1st and 3rd quadrants.		
Heavy Duty left 209014	For removal of supra- and subgingival calculus, in particular in the 2nd and 4th quadrants.		
Universal 209080	For removal of supra- and subgingival stain and calculus in all areas. For general scaling for those patients with moderate plaque or calculus.		

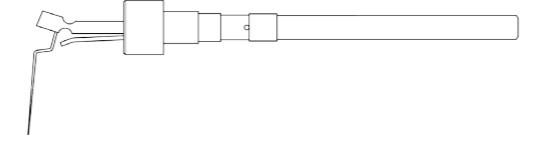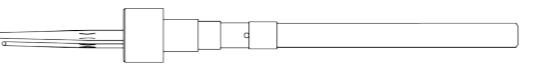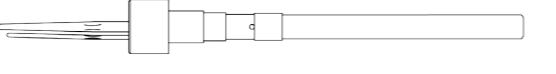ENDODONTIC TREATMENT

Endo 209070	Ultrasonic cleaning and canal preparation. Provides smooth canals and easy introduction of Gutta Percha. Can only be used with XO ODONTOSON 360 Endo files, ISO sizes 15, 25 & 40.		
Retro-instrument right 209096	For retrograde filling.		
Retro-instrument left 209093	For retrograde filling.		

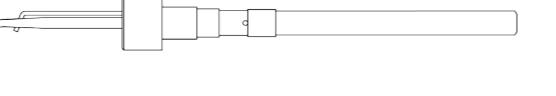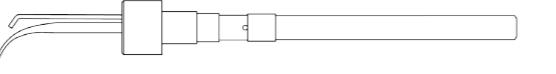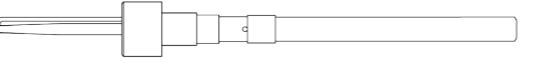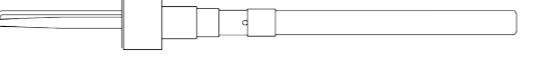PERIODONTIC TREATMENT

Perio 209030	Removes supra- and subgingival stain and calculus in all areas. Particularly useful for removal of calculus in pockets up to 13 mm deep. Can be used with antibacterial irrigants to provide reduction of depth of pathological pockets. Use the tip as if using a periodontal probe. REMEMBER: keep the tip moving with short strokes at all times.		
Thin Line straight 209034	Provides excellent furcation access in the 1st and 3rd quadrants. Similar tactile feeling as with a periodontal probe. To be used only after gross debridement with Perio instrument.		
Thin Line right 209036	Provides excellent furcation access in the 1st and 3rd quadrants. Similar tactile feeling as with a periodontal probe. To be used only after gross debridement with Perio instrument.		
Thin Line left 209038	Same characteristics and applications as Thin Line, right. Particularly suitable in the 2nd and 4th quadrants.		

Other languages available at xo-care.com.

All rights reserved.
XO® is a registered trademark.
© XO CARE 2017.

Products shown in this catalogue
are intended for use by dental professionals
for human dentistry procedures.
See updated technical and legal product
specifications at xo-care.com.
Subject to change.

XO CARE A/S

Copenhagen
Denmark

Usserød Mølle
DK 2970 Hørsholm
+45 70 20 55 11
info@xo-care.com
xo-care.com